

MAINTENANCE

The **Acadia** Amalgam Separator requires minimal maintenance. The only maintenance requirements include:

- Checking the Large Particle Inlet Filter Strainer for any blockages and cleaning as necessary.
- Checking the fill level of Solid Collection Filter and replacing as necessary.
- Replacing the Solid Collection Filter 12 months (one year) from the date of installation.

TROUBLESHOOTING

Problem	Possible Cause	Solution
Low suction or No suction.	a. Large Particle Inlet Filter Strainer clogged. b. Disposable Solid Collection Filter filled. c. Check that the input and output line connections to the Separator are tight. Make sure that there is no clogs or kinking collapsing.	a. Clean Inlet Filter Strainer. b. Replace the Solid Collection Filter. Call your authorized dealer if necessary. c. Call your authorized dealer for repair service if faulty connections are found.

WARRANTY

This Air Techniques equipment is warranted to be free from defects in material and workmanship for a period of twelve months from the date of installation by authorized Air Techniques Dealer service personnel.

Any item returned to our factory during the warranty period through an Authorized Dealer will be repaired or replaced at our option at no charge provided that our inspection shall indicate it is defective. Dealer labor, shipping and handling charges are not covered by this warranty.

This warranty does not apply to damage due to shipping, misuse, careless handling or repairs by non-authorized service personnel. The warranty does not include the Solid Collection Filter Replacement. Warranty void if installed or serviced by other than authorized service personnel. Air Techniques is not liable for indirect or consequential damages or loss of any nature in connection with this equipment.

This warranty is in lieu of all other warranties expressed or implied. No representative or person is authorized to assume for us any liability in connection with the sale of our equipment.

ON-LINE WARRANTY REGISTRATION

Quickly and easily register your new In-line Amalgam Separator on-line. Just have your product model and serial numbers available. Then go to the Air Techniques web site, www.airtechniques.com, click the **register a product link** and complete the registration form. This on-line registration ensures a record for the warranty period and helps Air Techniques keep you informed of product updates and other valuable information.

IF YOU NEED ASSISTANCE

Air Techniques Vacuum systems and accessories are designed and manufactured to high standards to deliver high-quality performance. If any difficulties are encountered with this product, please contact Air Techniques Technical Support as follows:

1-800-247-8324

Monday through Friday 8:30 a.m. – 5:00 p.m. Eastern Time

or

1-800-822-2899

Monday through Friday 8:30 a.m. – 5:00 p.m. Pacific Time.

For additional information, contact your authorized dealer or visit our web site, www.airtechniques.com.